USER MANUAL

Model: FL-100

English	2
Swedish - Svenska	20
Danish - Dansk	38
Dutch - Nederlands	56
Finnish - Suomi	74
French - Français	92
German - Deutsch	110
Greek - Ελληνικά	128
Norwegian - Norsk	146
Português: Do Brasil	164
Português: Europeu	182
Spanish - Español	200

Start by downloading the app: Flow - Depression treatment

Get started by downloading the app
“Flow - Depression treatment”

Table of Contents

1. Manufacturer	4
2. Intended Use	4
3. Box Contents & the Headset	5
4. Accessories	5
5. Contraindications	6
6. Precautions	6
7. Warnings	7
8. Possible Adverse Reactions	8
9. Prerequisites	10
10. The Treatment Program	10
11. First-Time Use	10
12. How to Perform a Treatment Session	11
13. Charging the Headset	14
14. Cleaning of the Headset	15
15. Device Specifications	16
16. Markings	17
17. Outside the EU	19
18. Product Disposal	19
19. Contact Us	19
20. EMC - Test Results	218

Instruction for Use – Flow FL-100

- Instructions on how to use the Flow headset and the accompanying smartphone app.
- IFU Version and Issuing date: Ver 19 - 2025-11-26
- Product model: FL-100.
- The latest version of this manual is also available to download from flowneuroscience.com

1. Manufacturer


Flow Neuroscience AB
Södra Tullgatan 3
211 40 Malmö
Sweden

2. Intended Use


- The Flow FL-100 is a Class IIa medical device intended to deliver a transcranial Direct Current Stimulation (tDCS) to the prefrontal cortex to treat mild to severe Major Depressive Disorder (MDD).
- Flow FL-100 is intended to be used by adults (18+) diagnosed with Major Depressive Disorder (MDD).
- Flow FL-100 is intended for home-based treatment and does not require any special training before use.

3. Box Contents & the Headset

- | | |
|-----------------------------|------------------------|
| 1. Instructions for Use | 7. Electrodes |
| 2. Micro-USB Charger & Cord | 8. Metal arch |
| 3. Two Pad Holder Rings | 9. Back piece |
| 4. Flow Headset | 10. Pads |
| 5. Button | 11. Charger port cover |
| 6. LED indicator | |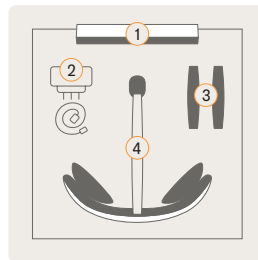

After each use, put the headset and other parts back in the box.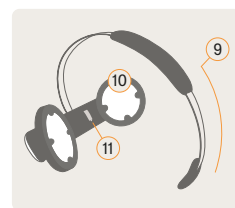

4. Accessories

- Headset Pads (Single Use)
- MicroUSB Charger & Cord

Reuse of pads may cause skin damage. Always start a stimulation session with a new pair of Headset Pads.

5. Contraindications

- Open wounds, broken skin, or damaged skin at the electrode sites.
- Metallic skull reconstruction at the electrode sites.

6. Precautions

The Flow device should only be used by people over 18 years of age who have been diagnosed with depression (Major Depressive Disorder).

If any of the following apply to you, talk with your doctor before using the headset:

- You are thinking of changing something about your current treatment, for example, changing your antidepressant dosage.
- You are unsure about your diagnosis.
- You are pregnant, think you might be, or become pregnant while using Flow.
- You currently have thoughts about suicide or harming yourself.
- You have been told there is a problem with your skull or if you have metal parts in or around the brain, like brain surgery clips, metal plates or screws in the skull, cochlear implants or brain pacemakers such as those used for Parkinson's disease. It is fine to use the device if you have dentures.
- You have epilepsy or have had seizures.
- You have a chronic skin condition affecting your forehead, such as psoriasis.

7. Warnings

- The headset should never be applied to open wounds, damaged or irritated skin. If the skin is still irritated or red since the last usage of the device, wait for the irritation to completely pass before applying the device again.
- New headset pads should be used each time, and these pads should not be dry.
- Never use tap water to wet pads.
- Apply the headset only on the head as directed and not on other body parts.
- When using the device, avoid activities which can cause the headset to move position.
- Keep the headset dry.
- Keep the headset out of the reach of children.
- The headset should not be used around strong magnetic fields, e.g. near an MRI machine.
- The headset is not indicated for the treatment of bipolar disorder.

8. Possible Adverse Reactions

- Some skin redness under the electrode or near the electrode sites is normal due to increased local blood flow. The redness will normally subside within approximately an hour after the end of use.
- DO NOT apply the device over open wounds, or damaged or irritated skin. This might create further irritation and/or skin damage.
- Skin irritation might be experienced beneath the stimulation electrodes. In this case, electrodes should NOT be reapplied to the irritated skin. If you experience any dryness of the skin/skin irritation after stimulation, it is recommended to apply moisturiser to the area.
- Discontinue treatment if the sensation of tingling, itching or burning under electrodes becomes severe (mild sensation of tingling, itching or burning is normal).
- Discontinue treatment if you experience persistent or worsening headaches.
- Discontinue treatment if you experience persistent tinnitus (noise or ringing in the ears) during or after using the device.

The following adverse reactions have been reported by users of Flow, grouped by incidence:

Incidence	Adverse Reaction
Common and Very common	Skin redness, itchy skin, stinging skin, skin irritation, headache
Uncommon	Skin pain, migraine, brain fog, memory disturbance, dizziness, worsening mood, worsening anxiety, insomnia, irritable, blurred vision, photopsia (flashing lights), tinnitus, eyelid twitching, nausea, vomiting, hyperphagia (always hungry), fatigue.
Rare	Aura, confusion, feeling high, paraesthesia (numbness), twitching muscles, restlessness, panic attack, nightmares, vivid dreams, aggression, suicidal ideation, eye sensitivity, eye pain, palpitations (heart racing), vertigo
Very rare	Skin burns, clumsiness, shaking/trembling, facial twitching, teeth clenching, neck pain, auditory hallucinations, hypomania, metallic taste, fainting

Incidence is defined as follows:

Incidence	Incidence rate
Very common	Greater than 1 in 10
Common	1 in 100 to 1 in 10
Uncommon	1 in 1000 to 1 in 100
Rare	1 in 10 000 to 1 in 1000
Very rare	Less than 1 in 10,000

NOTE: Any serious incident that has occurred in relation to the device should be reported to Flow Neuroscience and the local competent authority.

If you have questions about adverse reactions, please talk with your doctor or contact Flow Neuroscience.

9. Prerequisites

A smartphone or other device running either Apple iOS 15+ or Android 9+.

10. The Treatment Program

- The Flow treatment programme consists of 15 sessions during the first 3 weeks, as shown in the table below.
- After this first treatment period, 1-3 sessions are recommended per week.
- The number of sessions you can do per week is restricted, and you can only do 1 per day.
- Each session lasts 30 minutes, plus a few minutes of preparation time.

WEEK	W1	W2	W3	W4	W5+
No. of sessions	5	5	5	1-3	1-3


11. First-Time Use

- Download the “Flow - Depression treatment” app to your mobile device.
- Create an account in the downloaded “Flow - Depression treatment” app using an email address you have access to and a new password of your choosing.
- If a card with an activation code is supplied in the box, you might need to enter it in the app before your first session.

NOTE: IT security measures, including protection against unauthorised access, are secured in accordance with GDPR regulations.


12. How to Perform a Treatment Session

- Ensure your forehead is free from any makeup or hair products before you start. Not doing so can result in skin damage.
- If you have a fringe, it is recommended that you use a hair tie to remove it from your forehead.
- Take out the headset from the box.
- Press the button on the headset and make sure the LED indicator starts blinking. If not, charge the headset.
- Make sure Bluetooth is enabled on your smartphone.
- Start the Flow app and log in with the email address & password you chose in Section 11 above.
- Follow the instructions in the app to start the stimulation.
- When told to, follow the instructions on how to prepare the headset, as below: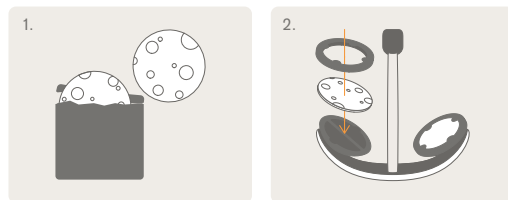


Take a new pair of Headset Pads from their sachet. Reuse of pads can lead to skin damage. Attach them to the electrodes using the Pad Holder Rings.

3.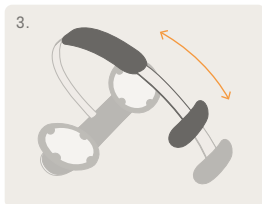


Adjust the length of the metal arch by sliding the back piece.

4.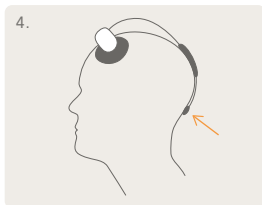

The end of the arch should sit comfortably on the back of your head. Make sure the headset does not sit too low.

5.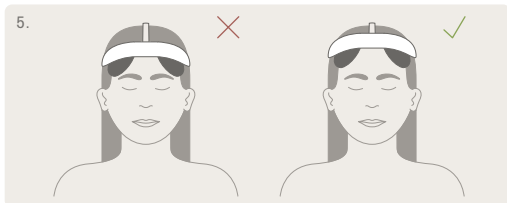


The app shows you how to correctly position the headset using the camera on your device.

6. Before starting the stimulation, you can press and hold the “Tingle” button in the app to test how the stimulation feels. Hold the “Tingle” button for at least 10 seconds.
7. Start the stimulation by pressing the “Start stimulation” button in the app.
 - The current will now slowly increase to its maximum of 2 mA, as shown in the app.
 - If you at any point want to pause the stimulation, press the pause button in the app or the button on the headset.
 - Follow the instructions in the app during the rest of the session. The session will last for 30 minutes.
 - When the time is up, the headset will automatically start decreasing the current.
 - Do not remove the headset before the app indicates that the stimulation has stopped.
 - The headset gives a “beep” signal when the stimulation has finished.
 - After the session, dispose of the Headset Pads and put the Pad Holder Rings and headset back in the box.
 - The headset will automatically turn off after a few minutes of inactivity. Start it again by pressing the button on the headset.

NOTE: Most users who experience reduced symptoms of depression do so within 3-4 weeks. For a summary of safety & clinical performance, see flowneuroscience.com

13. Charging the Headset

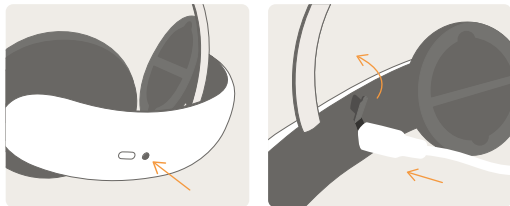- Charge the headset if the LED indicator on the front of the headset does not blink when pressing the button.
- Remove the charger port cover and charge using the supplied Micro-USB charger and cable.
- The LED indicator blinks during charging and shows a steady light when the headset is fully charged.
- Recharging a fully flat battery takes approximately an hour.
- Be sure to close the charger port cover again after charging.

14. Cleaning of the Headset

- It is recommended to clean the headset and especially the electrodes after each use.
- To clean the device, ensure that the Micro-USB cable is not connected to the headset and remove the pads.
- Use an IPA (isopropanol) wet wipe (45% is a good choice).
- Clean the electrodes using a gentle circular motion.
- Neither the device nor its accessories are supplied sterilised nor require sterilisation prior to use.

15. Device Specifications

PERFORMANCE CHARACTERISTICS	
PARAMETER	NOMINAL VALUE
MAX. STIMULATION CURRENT	2.0 + 0.05 mA
MAX. STIMULATION VOLTAGE	24 V
MICRO-USB CHARGER	5 V D.C.
BATTERY	RECHARGEABLE LiPo 3.7 V; 250 mAh
ELECTRODE SIZE	22.9 cm ² (5.4 cm diameter)
DEVICE WEIGHT	~110 g
WATERINGRESS PROTECTION	IP22
DEVICE EXPECTED LIFETIME	3 YEARS
OPERATING ENVIRONMENT	HOME OR CLINICAL ENVIRONMENT
OPERATING TEMP. RANGE	5 to 35°C (41 to 95°F)
STORAGE/TRANSPORT TEMP. RANGE	-10 to 40°C (14 to 104°F)
OPERATING HUMIDITY RANGE	10% to 93%, NON-CONDENSING
STORAGE/ TRANSPORT HUMIDITY RANGE	10% to 93%, NON-CONDENSING
OPERATING ALTITUDE RANGE	0 – 3000 m
STORAGE/TRANSPORT ALTITUDE RANGE	0 – 3000 m

16. Markings

ICON	DEFINITION
	Read the Instructions for Use
	Consult the Instruction for Use before operation
	Caution: Consult Accompanying Documentation
	Separate collection for electrical and electronic equipment per EC Directive 2012/19/EU. Waste Electrical and Electronic Equipment (WEEE).
	Type BF Applied part
	Manufacturing Year
	Medical Device
	Keep away from children
	Name of the device: FL-100
	Keep Dry
	Keep out of direct light
	Manufactured by Flow Neuroscience AB Södra Tullgatan 3, 211 40 Malmö, Sweden, Web: flowneuroscience.com
IP22	Water ingress protection: 22

ICON	DEFINITION
	EU Notified Body No.
	UK Approved Body No.
	Bluetooth Low Energy
	Expiry Date (Pads only)
	Lot Number
	Single Use Only (Pads only)
	Unique Device Identifier
	Contains integrated Lithium Ion Battery

17. Outside the EU

UK Responsible Person (UKRP)

Flow Neuroscience Ltd
4.05, 60 Grays Inn Road
London WC1X 8LU
United Kingdom

Swiss Authorised Representative (AUTHREP)

Neurolite AG
Hühnerhubelstrasse 79
3123 Belp
Switzerland

18. Product Disposal

- Headset Pads are cellulose sponges and can safely be disposed of as household waste.
- The headset and charger should be disposed of as electronic waste according to the local legislation.

19. Contact Us

If you have any questions regarding Flow, don't hesitate to contact customer support at <https://www.flowneuroscience.com/support/>

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Flow Neuroscience AB
Södra Tullgatan 3
211 40 Malmö
Suecia

20. EMC - Test Results

The device does not emit any radiation intended for medical purposes, neither hazardous nor potentially hazardous radiation.

EMISSION LIMITS & MEASUREMENT				
PHENOMENON	STANDARD	TEST METHOD	TEST LEVELS	RESULT
RADIATED RF EMISSION	IEC 60601-1-2	CISPR 11 ETSI EN 301 489-1	30~1000 MHz; 1~3 GHz & 3~6 GHz (Measurement Uncertainty 4.24 dB)	PASS
CONDUCTED RF EMISSION	IEC 60601-1-2	CISPR 11 ETSI EN 301 489-1	0.15~30 MHz (Measurement Uncertainty 3.35 dB)	PASS
VOLTAGE FLUCTUATION & FLICKER	IEC 60601-1-2	IEC 61000-3-3	EUT VALUES & LIMITS Pst: 0.028; Limit 1.00 Plt: 0.028; Limit 0.65 dc [%]: 0.0000; Limit 3.30 dmax [%]: 0.053; Limit 4.00 dt [s]: 0.000; Limit 0.50	PASS

IMMUNITY LIMITS & MEASUREMENT				
PHENOMENON	STANDARD	TEST METHOD	TEST LEVELS	RESULT
ELECTROSTATIC DISCHARGE	IEC 60601-1-2	IEC 61000-4-2	±2 kV, ±4 kV, ±6 kV, ±8 kV - Contact Discharge ±2 kV, ±4 kV, ±8 kV, ±15 kV - Air Discharge	PASS
RF ELECTROMAGNETIC FIELDS	IEC 60601-1-2	IEC 61000-4-3	10 V/m 80 - 2700 MHz 80 % AM at 1 kHz	PASS
ELECTRICAL FAST TRANSIENTS COMMON MODE / BURSTS	IEC 60601-1-2	IEC 61000-4-4	±2 kV for A.C. Power Lines 100 kHz Burst Frequency	PASS
SURGES LINE TO LINE	IEC 60601-1-2	IEC 61000-4-5	±0.5 kV & ±1 kV At 0°, 90°, 180°, 270° ; Repetition Rate: 60 S	PASS
SURGES LINE TO GROUND	IEC 60601-1-2	IEC 61000-4-5	±0.5 kV, ±1 & ±2 kV At 0°, 90°, 180°, 270°	PASS
RF COMMON MODE	IEC 60601-1-2	IEC 61000-4-6	3 & 6 V for A.C. Main Port; 80%, 1 kHz Amplitude Modulation Frequency: 150 kHz to 80 MHz	PASS
VOLTAGE DIPS & INTERRUPTIONS	IEC 60601-1-2	IEC 61000-4-11	0 % UT; 0.5 Cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270°, 315° 0 % UT; 1.0 Cycle 70 % UT; 25/30 Cycles	PASS